

VisiWrapTM

ROBOT




FACE PROTECTION

For ROBOTIC & TRENDELENBURG SURGERY

Visibility. Innovative. Safety. Implementation.

VisiWrap Robot- A safe and easy alternative to the mayo stand for Robotic and steep Trendelenburg procedures to protect the face from instruments.

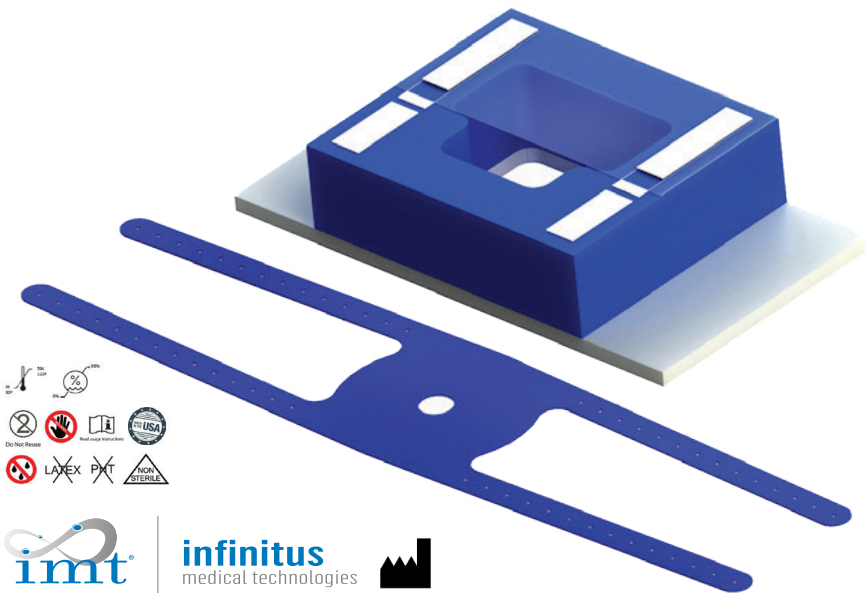

infinitus
medical technologies




7304 Vanclaybon Road, Apex NC 27523



919-285-1178
Infinitusmedical.com

Re-order P/N VWP-1000, 10 per case

STEP 1

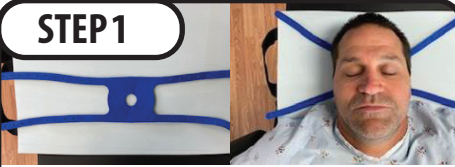

Place the anesthesia mask straps under patient's head. Blue side up.

STEP 2

Use the mask straps to help preoxygenate your patient as indicated.



STEP 3



Place VisiWrap Face Protector on patient's face with GREY foam touching face and BLUE side up.

STEP 4

Attach each of the 4 mask straps to the hook strips on the topside of the VisiWrap™.



STEP 5



Insert ETT, OG tube or an esophageal temp probe into the foam slits on either the left or right foam slits on the VisiWrap.

STEP 6

Place eye protection film via the hook and loop strips.



Thank you for your hard work and dedication to patient safety.